

Pairing Amplifon hearing aid with the Smart.5

Step-by-Step:

1. Initiate the pairing within the amplifon application

Pairing hearing aids

Restart your hearing aids and tap "Continue"

Pair your hearing aids

- Track and improve your usage
- Get personalised sound programs
- Access advanced settings

Turn on Location Services

You need to turn on Location Services to pair and locate your hearing aids, and locate your nearest store.

Need help? Select device type:

- Rechargeable
- Removeable battery

Pair now **Turn on Location Services** **Continue**

[Skip pairing](#) [Skip pairing](#) [Skip pairing](#)

2. Press the buttons on the hearing aid for around 3 seconds until a blinking green light appears

Pairing hearing aids

Select hearing aids

LE_L-Amplifon Hörgerät **>**
LE_Herr Stefan
AMPLI-ENERGY R 5

Left ✓ Right ✓

Don't see your hearing aids?
[Search again](#)

3. After a few seconds the hearing aid should be shown on the Smartphone, please press the arrow on the display

[Skip pairing](#)

4. „Pair & Connect“ will pop up twice on the top of the display, press it twice, give all permissions and accept
5. You can take out the hearing aid from the shell while the smartphone is loading
6. Please skip the Streaming section, it will be set up later
7. After that, put the hearing aid back in the shell and press the buttons for around 5 seconds in order to turn the device off
8. On the Smart 5, navigate to the Android system settings → Bluetooth and press „Pair new device“
9. Now press the buttons on the hearing aid for around 3 seconds until they are blinking green and take it out of the shell
10. The hearing aid should now appear as a “headset” in the results of the Smart.5
11. Press on it, give all permissions and press “Pair”
12. The hearing aid is now fully connected with the Smartphone and you can use the app control, the streaming function and the call functions



